

The Delasco CNH[®] Ear Protector



Order yours today!
Safe · Comfortable · Relief



Dermatologic Lab & Supply, Inc. • 1-800-831-6273
608 13th Avenue • Council Bluffs, Iowa 51501 USA
www.delasco.com/consumer

The Delasco **CNH® Ear Protector** is useful to relieve pressure on the ear after ear surgery or to relieve discomfort associated with the condition known as Chondrodermatitis Nodularis Helicis (CNH) - also known as painful nodule(s) of the ear.

PROTECT & RELIEVE

The Delasco **CNH Ear Protector** covers your ear and protects it from inadvertent pressure while turning in your sleep. The wide, comfortable elastic band eases pressure on the head. The generously sized cavity can accommodate all ear sizes. *Not intended for sustained, direct pressure on the affected ear but only for inadvertent pressure such as when sleeping.*

CUSTOM FIT

The strap is attached to the ear protector by means of Velcro at each end. Since there are 4 possible attachment points on the ear protector, there are 6 possible combinations of attachments to assure the most comfortable fit. The 3 most comfortable attachments are shown at right. *The CNH can also be attached using the bottom velcro points in 3 similar positions.*

SAFE & COMFORTABLE

The **CNH** provides the safest, most customized form of treatment. It is made from specially designed foam that is flexible yet very dense in order to support and elevate the head. Both the strap and ear protector are washable in mild soap with air drying (no heat). Latex-free.

Contact Delasco for more information on how you can order the **CNH Ear Protector**. We hope you are delighted with this product!



*Top and bottom attachment —
most comfortable*



Top and top attachment



Top and crossover attachment

Dermatologic Lab & Supply, Inc.

608 13th Avenue, Council Bluffs, IA 51501 USA

Toll-Free Phone: 1-800-831-6273 • Phone: 1-712-323-3269

Toll-Free Fax: 1-800-320-9612 • Fax: 1-712-323-1156

Email: questions@delasco.com • Web: www.delasco.com/consumer

